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Dear customer

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General product description

Herbst TS is the latest development – a construction with a telescope mechanism in three parts. This design provides several key treatment advantages:

- a design with three integrated parts connected together and which cannot separate, eliminating the risk of rod to sleeve disengagement during treatment.
- the lower screw and pivot (F-base) are positioned central to the first bicuspid and the upper screw and pivot (F-base) are positioned central to the first molar, thus reducing irritation of the mucosa.
- the use of special screws and F-bases enables lateral movement of the patient's lower jaw.

Herbst TS is available in three lengths; 20 mm, 22 mm and 24 mm. The length is measured from the center of the upper eyelet to the center of the lower eyelet. With traditional Herbst appliances the length of the sleeve and/or rod must generally be reduced to accommodate the patient's specific requirements, whereas with the Herbst TS no length reduction is required.

With the telescope mechanism, the laboratory technician positions the lower base central to the first bicuspid, and places the upper base in the center of the first molar. These key placement points allow the operator easier access to the screws, and reduces the overall bulk of the appliance once seated in the patient's mouth.

The upper and lower eyelets have been manufactured with an offset. This design feature provides greater patient comfort since the head of the screw is in line with the rod of the telescope mechanism.

Since Herbst TS cannot be separated to allow traditional spacer rings to be slipped over the Herbst rod, simply place and crimp our slotted spacer rings on the lower section of the rod to continue mandibular advancement during the course of treatment. This technique is easier and faster than the traditional method. Crimpable spacer rings and clamping pliers for Herbst TS are available.

3. Fabrication and application of the Herbst TS appliance

Pour the impressions, which include the waxed out bands fitted on the first premolars and molars, with dental stone (e.g. Rapidur®). Mount the models in the correct position in an articulator with plaster using the construction bite recorded by the operator. Bend and laser weld the half-round remalay® wire (ø 0.75 x 1.5 mm) or round remalay® wire (ø 1.0 mm) to the bands to form a lingual arch. A palatal bar can be placed in the upper jaw between the first molars to provide stabilization.

Next laser weld the bases of the hinge units to the bands. Ensure that the screws are at the correct angle to the axis between the upper and lower jaw. It is also important that the hinges between the upper and lower jaw operate without friction to ensure smooth functioning of the telescope mechanism. Friction can damage the appliance – especially the bands.

The telescope tubes and telescope rods cannot be adjusted individually with the Herbst TS retention hinge. The matching retention hinge is selected according to the construction bite by closing it and placing it on the base or by measuring the distance between the two bands beforehand.

Notes:

"Double bands" should be used to minimize the risk of the bands being damaged by shear forces. This involves welding a second band over the band already fitted (e. g. using the spot welding unit Master 3000 or laser).

Generally the second band is 2 to 3 sizes larger than the first band fitted. Ready-welded "double bands" can be ordered from Dentaurum. The order form is available on request.

Fitting intra-orally:

The bands are fitted intra-orally using standard band cement. We recommend covering the bases of the hinges with wax to ensure that the cement does not penetrate into the thread. After curing of the cement the Herbst TS is unscrewed.

If necessary, the Herbst TS appliance can be activated using spacer rings (see product range).

Make sure that the appliance can be removed quickly in case of a medical emergency, if necessary.

4. Composition

The composition is included in the materials list; please refer to the catalog or www.dentaurum.com.

5. Safety instructions

The product should not be used if there is a known allergic reaction to one or more of the material components. The fit of the Herbst TS hinge and tightness of the screws should be checked at regular intervals to prevent the appliance from spontaneously coming apart and the components being swallowed or aspirated! If necessary, tighten the screws and secure them with a special screw securing adhesive, e. g. Ceka® Bond. The loops of the Herbst TS must not be bent. It is not intended that the loops be ground.

6. Information for single use products

The Herbst TS retention hinge is intended for single use only. Reconditioning of Herbst TS that has already been used (recycling) and its reuse on a patient are not permitted.

7. Product range

Herbst TS retention hinge

Telescope system
For welding or soldering to bands or cast splints.
Length: 20 mm REF 607-126-20 1 set
Length: 22 mm REF 607-126-22 1 set
Length: 24 mm REF 607-126-24 1 set

Contents:

- 1 x telescope attachment for right
- 1 x telescope attachment for left
- 4 x hexagon socket screws
- 4 x F-base
- 1 x Instructions for use

An Allen key is not included in the delivery.

Allen key REF 607-144-00 1 piece
for Herbst TS I/II/SUS
sterilizable
Size 1.5

Herbst F-base REF 607-127-30 10 pieces
for Herbst TS + I retention hinges

Herbst TS spacer rings, slotted

Length: 1 mm REF 607-103-11 10 pieces
Length: 2 mm REF 607-104-11 10 pieces
Length: 3 mm REF 607-105-11 10 pieces
Length: 4 mm REF 607-107-11 10 pieces

Herbst TS clamping pliers REF 003-710-00 1 piece
sterilizable

For clamping slotted spacer rings for the Herbst TS retention hinge.
Premium Line – 10-year guarantee

8. Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

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
9. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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