
Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

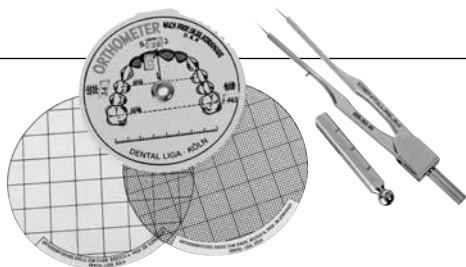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Dear customer

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application.

In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaureum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaureum GmbH & Co. KG

Turnstr. 31 | 75228 Ispringen | Germany

2. General product description

The Orthometer kit consists of:

- orthometer
- bow divider with sleeve
- centimeter disc and millimeter disc
- case

Orthometer:

The device consists of a central white plastic disc with an engraved scale; this rotates easily between two windowed outer scales. When the instrument is set at the total width of the incisor teeth (S.I.), it is possible to read off the relevant values for the transverse and sagittal dimensions for both the upper and lower dental arches simultaneously.

Bow divider with sleeve:

Small precision dividers for measuring tooth width. The bevelled sleeve has an engraved mm-scale which can be used to measure the sagittal anterior tooth gradient.

Centimeter disc and millimeter disc:

Transparent plastic discs with engraved scales for the analysis of casts and profile photographs.

Case: For storing the Diagnostic Instrument acc. to Korkhaus.

3. Application

Using the dividers, the maximum width of the four upper incisors is measured and added together = sum of the incisors (S.I.). The middle disc on the orthometer is then turned until this value can be seen in the section S.I. The following data, stipulated by Korkhaus as reference values, can then be read from the other windows on the instrument:

- a) the anterior arch width, i.e. the transverse distance between the first bicuspid.
- b) the posterior arch width, i.e. the distance between the first molars.
- c) the anterior arch length (l), i.e. the median line that runs at a right angle from the line connecting the first bicuspid to the labial surface of the central incisors.

A comparison with the measured arch lengths and widths will indicate the required course of treatment. The values of the orthometer are based upon the values given by the Pont index, modified by Korkhaus.

4. Availability

Diagnostic instrument acc. to Korkhaus, 1 set	REF 029-360-01
Orthometer, 1 piece	REF 029-364-00
Bow divider with sleeve, 1 piece	REF 029-361-01
Centimeter disc, 1 piece	REF 029-362-00
Millimeter disc, 1 piece	REF 029-363-00

5. Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is responsible for the processing of the products. Responsibility for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

6. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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